

Yes!Star 

# Yes!Star Dental X-Ray Film



Shanghai Yestar Co., Ltd



## Product Series

| Product Name                | Type                    | Size     | Specification                                    | Package      |
|-----------------------------|-------------------------|----------|--|--------------|
| Yesstar GY-D( Bright Room)  | Speed D                 | 30×40mm  | 100pcs/box                                       | 80box/carton |
| Yesstar GY-E( Dark Room)    | Speed E                 | 30×40mm  | 150pcs/box                                       | 80box/carton |
| Yesstar Monobath GYM- II    | Liquid(For Bright Room) | 250ml    | 250ml/btl  | 40btl/carton |
| Yesstar Lateral Cephalogram | Green Sensitive Film    | 8×10inch | 50pcs/box  | 10box/carton |
| Yesstar Panoramic Film      | Green Sensitive Film    | 6×12in   | 50pcs/box  | 10box/carton |
| Yesstar Dental Film Set     | Speed D                 |          | 50pcs bright room films<br>250ml monobath syring | 50box/carton |



# Wash Dental Film In Darkroom(Speed E)

## Wash Condition and Method

1. Choose dental film wash machine: 3-5 minutes.
2. Choose manual operation: washing liquor temperature 18 °C - 20 °C, developed for 3-5 minutes, fixed for 10 minutes, wash for 30 minutes.

## Attentions

1. There is a convex point on the horns of dental film bag, used to judge everyone's left or right teeth. And the convex point side indicates the side faced radiation source.
2. Darkroom wash: Film should be unpacked in the dark room under safe-light (red light), exposure time should not be too long and flush as soon as possible to avoid affecting film quality.
3. Don't make developer solution and fixing solution splash eyes, if splashed please immediately wash with water until painless.



## Exposure Conditions

Focal Distance 20cm(8in) , 70KV,7mA

| Upper Teeth |     | Time  |
|-------------|-----|-------|
| 321         | 123 | 0.15s |
| 654         | 456 | 0.20s |
| 87          | 78  | 0.25s |
| Lower Teeth |     | Time  |
| 321         | 123 | 0.13s |
| 654         | 456 | 0.14s |
| 87          | 78  | 0.15s |



# Wash Dental Film In Lightroom(Speed D)

## Wash Condition and Method

1. Film the dental film by normal procedure.
2. Extract 3-5ml monobath with injector, inject from the upper right corner into the dental film bag.
3. Press on the pinhole, shake or knead the dental bag properly to coat the monobath on the film evenly, tear off film bag after 60-90s, and rinse off with water. (Note: Do not use your fingers to wipe the film's surface in case of scratched,)
4. Dry the film off for inspection and diagnosis.



Lightroom Boxset

## Exposure Conditions

Focal Distance 20cm(8in) , 65KV,1.5mA

| Service Life | Anterior Teeth Exposure Time | Carine Exposure Time | Posterior Teeth Exposure Time |
|--------------|------------------------------|----------------------|-------------------------------|
| 1-3 years    | 0.6s-1.0s                    | 0.6s-1.0s            | 0.7s-1.2s                     |
| 3-5 years    | 0.7s-1.2s                    | 0.7s-1.2s            | 0.8s-1.5s                     |
| ≥5 years     | 0.8s-1.4s                    | 0.8s-1.4s            | 0.9s-1.5s                     |

[ This is for your reference only. ]

Focal Distance 20cm(8in) , 70KV,8mA

| Upper Teeth |       | Time  | Lower Teeth |      | Time  |
|-------------|-------|-------|-------------|------|-------|
| 321         | ↑ 123 | 0.4s  | 321         | 123  | 0.34s |
| 654         | 456   | 0.54s | 654         | 456  | 0.34s |
| 87          | ↓ 78  | 0.6s  | 87          | ↓ 78 | 0.4s  |



# Wash Dental Film In Lightroom(Speed D)

## Attentions

1. There is a convex point on the horns of dental film bag, used to judge the left or right teeth. And the convex point side indicates the side is faced radiation source.
2. The temperature of using disposable monobath is good to set as 22°C, higher temperature makes the image too black and formed fog easily, lower makes the image too quiet. Users can adjust the temperature according to the situation until clear.
3. Should be careful to inject liquor with a syringe, avoid transpierce the dental film bag to cause leakage.
4. Need to use the assorted Yes!Star monobath. Please seal monobath after use.
5. Avoid monobath splash into eyes, rinse with water immediately if contacted.





# Dental Panoramic Film

## Wash Condition and Method

1. Film should be unpacked in the dark room or changing bag (box), and load into the film cassette timely.
2. Follow normal procedure to shoot the film.
3. After shooting, the film should be developed by a high temperature flush technique in the automatic film processor timely.

## Attentions

1. Panoramic film must in combination with green sensitive intensifying screens.
2. After unsealing, the untapped films have to pack in dark room or changing bag (box) to keep out of the sun.
3. Storage Conditions: Room temperature about 10-23℃, relative humidity about 30-65%.





# Enterprise Introduction

## Shanghai Yestar Co., Ltd

Yes!Star is Captial Group's investment company in China. Until now Yes!Star has set up 26 branches, and makes marketing network in whole country. The main business is Fuji Film image product sales and after-sale services. Yes!Star is unique general agent of Fuji Film civil image products in China. In 2011, Yes!Star follow to launch some new products, including Yes!Star brand industrial x-ray film, Yes!Star brand dental x-ray film, Fuji medical digital laser film ,etc. And then we expand to professional sensitive consumables manufacture and sales in medical and industrial area. Belong with Capital Group, Guangxi Yes!Star Co., Ltd. is a manufacturing enterprise, which owns advanced technique and equipment imported from Japanese. It adopts Fuji strict quality control system, and builds professional research and examination team. Guangxi Yes!Star Co., Ltd is worthy of the name that modernization image materials production base. At present, Yes!Star has developed to one of the leading company in domestic image industry.

Web: [www.yesstarnet.com.cn](http://www.yesstarnet.com.cn)

Add: 8th Floor ,Building A, Greenland Tower ,No.58 Xing Jian Dong Road, Shanghai, China

Tel: 021-64890111      Fax: 021-64890555

Yes!Star 

Pursuit of Perfect Quality